





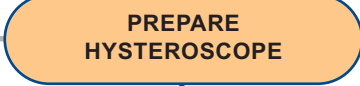



| INSTRUMENTS MATERIALS | PRE-PROCEDURE | KEY POINTS | WARNINGS / PRECAUTIONS |
|---|---|---|--|
| |  | <ul style="list-style-type: none"> - Assess decision for permanent contraception - Counsel patient on risks and benefits - Review for contraindications - Perform screening tests & pelvic exam - Schedule procedure during early proliferative phase - Discuss Post-procedure recovery | <ul style="list-style-type: none"> - Patient should understand that Essure is permanent and irreversible - 3 months alternative contraception is required (IUD cannot be used) |
| |  | <ul style="list-style-type: none"> - Reconfirm patient's decision for permanent contraception - Discuss mandatory requirement for alternative contraception until 3 month follow-up visit - Obtain informed consent - Discuss failure rate - Discuss pain expectations | <ul style="list-style-type: none"> - Patient should arrange for transportation post procedure |
| <ul style="list-style-type: none"> - Informed Consent Form |  | <ul style="list-style-type: none"> - Conduct pregnancy test within 24 hours prior to but preferably immediately preceding the micro-insert placement procedure | <ul style="list-style-type: none"> - Pregnancy test must be within 24 hours |
| <ul style="list-style-type: none"> - Pre-procedure medications |  | <ul style="list-style-type: none"> - Administer non-steroidal anti-inflammatory drug one to two hours before the procedure - If using only para-cervical block, Diazepam or similar agent may be offered 30 minutes before the procedure | <ul style="list-style-type: none"> - Discuss with patient that additional anesthesia may be given if requested - Administer pre-procedure medications with enough time for them to take effect |
| <ul style="list-style-type: none"> - Open sided or detachable Sims speculum - Tenaculum - Gauze & forceps - Antibacterial prep solution |  | <ul style="list-style-type: none"> - Inspect Equipment - Place patient in lithotomy position - Introduce speculum - Prep cervix with anti-bacterial solution | <ul style="list-style-type: none"> - Ensure emergency equipment and supplies are available - Observe sterile technique throughout procedure - Observe universal precautions |
| <ul style="list-style-type: none"> - Hysteroscope - Camera - Camera head - Light Source - Light Cable - Sealing Cap - Monitor - Inflow with Y connector - 2x2 liter or 1x3 liter warmed saline bag |  | | |
| |  | <ul style="list-style-type: none"> - Connect camera, light source, sealing cap, fluid inflow and outflow tubing to hysteroscope - Focus hysteroscope - Perform white balance - Test Fluid Flow | <ul style="list-style-type: none"> - Warm saline bags are available - Have multiple saline bags immediately available - Ensure that camera and proximal scope lens are dry |
| <ul style="list-style-type: none"> - Tenaculum |  | | |